

Delivery Cannula

INDICATIONS FOR USE

The Delivery Cannula is a single-use, sterile delivery system used to provide a more precise application of demineralized bone product and better access to remote surgical sites.

WARNING

- Do not re-sterilize and/or reuse. The Delivery Cannula is for single-use only.
- Do not use damaged and/or expired product.

DEVICE DESCRIPTION

The Delivery Cannula consists of a pusher and a delivery cannula.



The Delivery Cannula is composed of Bomed HD810MO, which is a polypropylene homopolymer material.

This material has successfully passed the biological tests according to ISO 10993-external communicating devices for indirect blood contact for a prolonged period.

CONTRAINDICATIONS

No contraindications are known to exist.

DIRECTIONS FOR USE

1. Ensure that packaging is not damaged and/or expired.
2. Prepare demineralized bone product (not included) and insert into syringe body (not included), if needed.
3. Apply tapered end cap (not included) to syringe, if needed.
4. Thread cannula onto tapered end cap.
5. Insert pusher onto the distal end of the syringe, or onto the distal attachment (not included).

GAMMA IRRADIATION TREATMENT

All medical devices are terminally sterilized using Gamma irradiation with a dose ≥ 15.8 kGy to ensure patient safety.

RETURN POLICY

Please contact Amendia Customer Service for medical device return criteria and a required return authorization number.

Amendia Account Services
(877) 755-3329
accountservices@amendia.com

MANUFACTURED FOR:

Amendia
1755 West Oak Parkway, Ste. 100
Marietta, GA 30062
(877) 755-3329